earWear

YOUR EARS. YOUR RULES.


Earwear Intro

Work | Calm | Party | Road | Music | Hunt | Aqua

Reusable Hearing Protection

CE 0511

EN 352-2

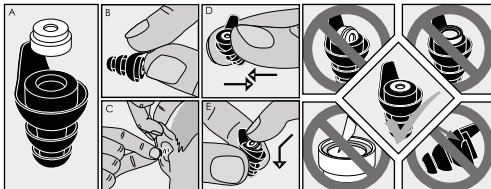Products are tested acc. to EN 352-2:2020+A1:2024
 Certification to the regulation EU 2016/425 was performed by PZT GmbH: Bismarckstraße 264 B, 26389 Wilhelmshaven, #1974, Germany
 The production control module D is performed by AUVA: Adalbert-Stifter-Straße 65, 1200 Wien, #0511, Austria

NEUROTH INTERNATIONAL AG declares that the hearing protection is in compliance with PPE regulation EU 2016/425. The full text of the EU declaration of conformity can be found at the following link: <https://www.neuroth.com/de-AT/gehoerschutz/>

Ear Tips

Sizes	M (Medium)	L (Large)
nominal diameters	6 - 11mm	7 - 12 mm

EN USER INFORMATION

The hearing protection consists of two parts; the pill shaped filter and the universal re-usable ear tip (Picture A). The filter is supplied fitted into the large ear tips (7-12mm). First insert this tip into the ear canal by following the instructions. If the tip is difficult to insert into the canal or feels uncomfortable then try the medium size ear tip (6-11mm). Don't forget to place and correctly position the filters in the ear tip (see "Changing the ear tips"). It is important that the tip seals correctly in order to achieve the certified attenuation.

Inserting into your ears

In order to reduce the risk of infection it is important that both your hands and the hearing protection are clean before insertion into ears. Grip the tongue of the ear tip between thumb and forefinger (Picture B). To ease insertion, pull the top of the ear outward and upward, this is best achieved by holding the ear with your opposite hand, your arm placed around the back of the head (Picture C). Gently push and twist the hearing protection into the ear until it sits comfortably in the ear canal whilst forming a good seal with the ear canal. Position the tongue at the top of the ear making it easier to find and grip when removing the tip from the ear.

Removing from your ears

WARNING: Sudden or fast removal of earplugs out of the ear canal may damage the ear drum. Remove the hearing protection with a slow twisting motion to gradually break the seal with the ear canal.

Changing the ear tips

To change the ear tips, you must first remove the filter (Picture A) from your existing ear tips.

To remove the filter;

Gently squeeze the ear tip just below the filter until it 'pops' out of the ear tip (Picture D).

To place the filter in ear tip;

- Ensure that the smaller side of the filter is facing upward so this side will face out of the ear tip once inserted.
- Insert the filter into the larger opening of the ear tip. This is achieved by inserting the filter at an angle, pushing the lower side of the filter in first. Then press down to correctly position the filter in the cavity of the ear tip (Picture E). In order to achieve the certified attenuation, the filter needs to be correctly fitted and positioned.

Cleaning the hearing protection

Clean your ear tip after each use to remove earwax (Cerumen) and other debris. The ear tips can be wiped clean with a damp cloth or antibacterial cleaning tissues. After removing the filter (see "Changing the ear tips"), the ear tips can be cleaned with soap or hygiene tablets and then rinsed in tepid water and dried thoroughly. Only if necessary the filter can be cleaned with soap or hygiene tablets. After cleaning the filter needs to be thoroughly rinsed in tepid water and allowed to dry slowly.

Storage

Always store your hearing protection in a clean, protective carry case. Do not store other objects therein.

Warnings

- Suitable earplugs must be worn at all times in noisy environments.
- The hearing protection must be fitted, adjusted and maintained in accordance with the instructions. Cleaning regularly and paying close attention to maintenance ensures sustained, optimum, performance.
- Before inserting the hearing protection into the ear canal, first ensure the filter is correctly positioned in the ear tip.
- To achieve the specified level of attenuation and hearing protection, the ear canal must be completely sealed by the ear tip.
- Regularly inspect the ear tips for tears or cracks in the flanges and body to ensure continued serviceability. Replace if necessary. For replacement contact manu-

facturer.

- If the above warnings and recommendations are not adhered to, the protection afforded by this product will be severely impaired.
- The hearing protection may be adversely affected when coming into contact with chemicals or alcohol.
- The hearing protection parts can be damaged when coming into contact with sharp objects.
- Keep the hearing protection out of the reach of young children.

Warranty

This product is covered by a two (2) year limited manufacturer warranty.

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Neuroth International AG
PZT GmbH, rozpoznavna številka 1974