
E A R W E A R

ADAPT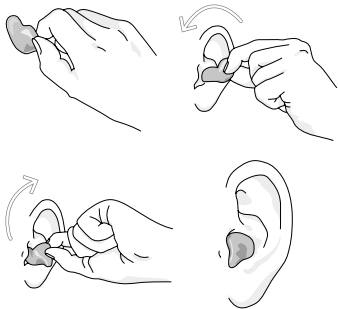

Dear customer,

please read these usage instructions attentively before using your earmould for the first time and keep them safely in the supplied storage case. They provide you with important information about your product.

Listening to music that is too loud with earphones can lead to severe hearing damage. Bear in mind that the health of your ears is very important. Protect your hearing by using earphones sensibly and limiting the volume to a safe level.

General information:

- ▶ ADAPT is a reusable, individually adjusted earplug made of flexible, skin-friendly material.
- ▶ To distinguish between the sides, the right earpiece features a dot.
- ▶ Note: When using ADAPT, acoustic perceptions of the surroundings may be restricted.

Insertion

Grasp your earpiece with the thumb and forefinger and place the cone of the earpiece into your auditory canal. Pull your earlobe downwards carefully and push the earpiece, if necessary, into the auditory canal, turning it gently. Follow the natural con-

tours of your ear. Press lightly on the earpiece to ensure it fits pleasantly in the ear.

Removal

Grasp the earpiece with your thumb and forefinger and pull it carefully so that it loosens from the auditory canal. Then remove the earpiece from your ear.

Functional interference

- ▶ Check your EARWEAR ADAPT regularly for damage. We would be happy to check whether your customised earpiece still fits perfectly.
- ▶ Do not use the earpieces if they are damaged. Please contact the manufacturer for replacements.
- ▶ To be able to ensure good wearing com-

fort/good hearing experiences, we recommend that you replace your customised earpieces at the latest after 4 years.

(Date of recommended replacement on the product label/case)

Cleaning and care

- ▶ Clean your customised earpieces carefully if dirty with mild soapy water.
- ▶ Your customised earpieces can be negatively influenced by certain chemical substances. For further information, contact the manufacturer.
- ▶ Ask your customer adviser for information about specially recommended care products.
- ▶ Please use the case provided for safe-

keeping before and after use.

Warnings

- ▶ The EARWEAR ADAPT is not hearing protection and does not provide protection against harmful ambient noise!
- ▶ Avoid listening to music with earphones at high volume for lengthy periods. Ensure that noise levels are moderate in order to protect your ears.
- ▶ Take regular breaks from listening to music to give your ears a rest and avoid potential damage.
- ▶ Please note that EARWEAR ADAPT compromises your perception of possible dangers or warning signals. When using them, therefore pay special attention to potential

dangers and warning signs in your surroundings.

- ▶ In case of pressure sores or a feeling of pressure, please check the fit of the earpieces.
- ▶ If allergic reactions occur such as reddening of the skin, itchiness etc., please discontinue use of EARWEAR ADAPT and clarify the cause with a medical consultant.
- ▶ Keep the product out of the reach of small children as it is a choking hazard!
- ▶ Sudden or rapid removal of an earpiece from the auditory canal can damage the eardrum.
- ▶ Do not remove the earpieces by tugging on the wearing strap or cable!
- ▶ In case of injury to the ear, the earpiece

may no longer be used and it is recommended to consult your medical practitioner.

- ▶ EARWEAR ADAPT should not be worn if there is a risk of the wearing strap or cable being snagged or caught during wearing.

Service and warranty

In case of a warranty claim, please contact your retailer with the receipt. The warranty terms are governed by statutory provisions. The warranty excludes damage due to improper use and non-compliance with the care instructions.

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Neuroth International AG