
E A R W E A R

**COMFORT**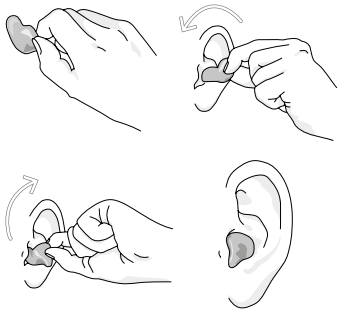

**Dear customer,**  
please read these usage instructions carefully before using your hearing protection for the first time and keep them to hand in the case. They provide important information about your product.

Protect your hearing because noise-induced hearing loss is incurable. Noise is unwanted sound that is not only an unpleasant nuisance but can also be the trigger for many illnesses. Noise-induced hearing loss no longer occurs just at work but is increasingly the consequence of leisure activities

detrimental to hearing. Unfortunately, the use of hearing protection is very often carelessly neglected. The reason for this is often that it is not comfortable to wear. The EARWEAR COMFORT provides safe protection for your hearing and excellent wearing comfort!

**General information:**

- ▶ The EARWEAR COMFORT is a reusable and individually adjustable earplug made of soft antiallergenic material.
- ▶ To differentiate which side is which, there is a colour marking on the right side.
- ▶ Note: Adequate protection is only assured if your hearing protection is used properly, fits securely and is worn continu-

ously when you are exposed to noise.

- ▶ The product has been tested and approved according to EN352-2:2020.
- ▶ The type approval test and the annual monitoring audit are carried out by Allgemeine Unfallversicherungsanstalt (General Accident Insurance Institution) – Technical Safety Department, appointed authority No.0511.
- ▶ Source details for declaration of conformity: <https://www.earwear.me/service/produkt Daten>
- ▶ Source details for harmonised standards: EUR-Lex - 52018XC0327(02) - EN - EUR-Lex (europa.eu)
- ▶ Source for the directive PSA-VO EU

2016/425: Personal protective equipment (PPE) (europa.eu)

### **Insertion**

Grasp the carry strap fixture or the nylon string between your thumb and index finger and insert the earplug into the auditory canal with gentle pressure. Insert the earplugs deeper into the auditory canal with a gentle rotation.

### **Removal**

► Grip option: Grasp the gripping section with your thumb and index finger and pull it carefully to release the earplug from the auditory canal.

► Option without grip: Grasp the hearing protection between your thumb and index finger and then draw it out of the ear.

### **Functional interference**

► Regularly check your EARWEAR COMFORT for damage and any required maintenance. Do not use them if they are damaged. Please contact the manufacturer for replacements. (6.3 h.)

► Your hearing protection product should be checked every two years by a specialist.

► We generally recommend replacing your hearing protection product at the latest 4 years from manufacture. (Date of recommended replacement on the product label/case)

## **Cleaning and care**

- ▶ Clean your hearing protection product carefully after each use with lukewarm water or with mild soapy water for more persistent dirt.
- ▶ This product can be negatively influenced by certain chemical substances. Please contact the manufacturer for further information.
- ▶ Ask your customer adviser for information about specially recommended care products.
- ▶ Please use the case provided for safekeeping before and after use.

## **Warnings**

- ▶ Your EARWEAR COMFORT only provides

adequate protection if it is used, adjusted and maintained in accordance with the instructions of the manufacturer.

- ▶ In case of pressure sores and a feeling of pressure, please check the fit of the hearing protection product.
- ▶ If allergic reactions occur, such as reddening of the skin, itchiness, etc., please do not continue to use the hearing protection and clarify the cause with a medical specialist!
- ▶ Keep the product out of the reach of small children as it is a choking hazard!
- ▶ Please note that the earplugs may impede your perception of possible dangers or warning signs. Therefore, when using your earplugs, pay special attention to pos-

sible hazards or warning signs in your surroundings.

- ▶ Abrupt or quick removal of the earplugs from the auditory canal can damage the eardrums.
- ▶ Do not remove the earplugs by tugging on the carry strap!
- ▶ In case of ear injuries, the earplugs must no longer be used and it is recommended to seek medical advice!
- ▶ The earplugs should not be worn if there is a risk that the carry strap might snag or be yanked during wearing.

### **Service and warranty**

In case of a warranty claim, please contact your retailer with the receipt. The warranty terms are governed by statutory provisions. The warranty excludes damage due to improper use and non-compliance with the care instructions.

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## Definition of terms

Values (dB)	Sound insulation in decibels, whereby for 84% of individual persons, the achieved protection is greater than or equal to the stated value.
APV (dB)	Assumed protective effect („Assumed protection value“). Average insulation value (Mf-1xSf) for the stated frequency in decibels.
Mean (dB)	Average insulation value in the stated frequency range (SNR/L/M/H) or the stated frequency ranges (frequency (Hz)) in decibels.
StD (dB)	Standard deviation for the stated sound insulation value in decibels
SNR (dB)	(„Single Number Rating“). Sound insulation value, to be deducted from the C-valued noise level.
L (dB)	Sound insulation value in a predominantly low-frequency noise spectrum (frequency<500Hz)

M (dB)	Sound insulation value in a predominantly medium-frequency noise spectrum (frequency >500Hz and frequency <2000Hz)
H (dB)	Sound insulation value in a predominantly high-frequency noise spectrum (frequency >2000Hz)

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Neuroth International AG
PZT GmbH