
E A R W E A R  
**DIVE**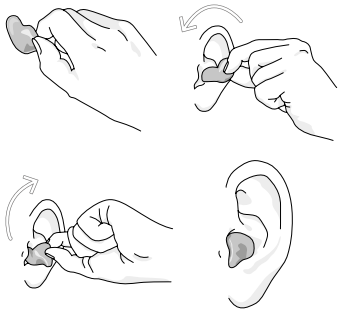

**Dear Customer,**

Please read this leaflet carefully before using your hearing protection Earwear DIVE for the first time and store it in your case. It provides important information on your product.

Earwear DIVE is highly comfortable to wear, and also ensures your comfort when diving, snorkelling, swimming or surfing.

**General information:**

- ▶ Earwear DIVE is a comfortable, customised earmould.
- ▶ It uses a small hole in the ear canal section to limit the amount of water entering your ear canal, which means the temperature in your ears stays at a comfortable warm temperature for longer. The slim design of Earwear EW 1521 also makes the product easy to wear under a diving hood if necessary.
- ▶ Earwear DIVE is customised for the ears of the purchaser and is therefore only suitable for use by him/her.
- ▶ When used correctly, Earwear DIVE will not impact on pressure equalisation.

**Use:**

Insertion: Take your Earwear DIVE between your thumb and index finger and insert it into your ear canal. Use gentle pressure to move it to its final position.

Removal: Hold the outer section with your thumb and index finger and pull carefully so that the earmould comes loose from the ear canal.

**Maintaining the functionality of your Earwear DIVE:**

- ▶ Check your Earwear DIVE regularly for any damage.
- ▶ Earwear DIVE should be checked in-ear by an expert on an annual basis, so as to ensure that they are fully effective.

► It is essential to ensure that your ear is free of earwax (cerumen). If it is not, the earwax may seal the aperture when you insert the earplug, and prevent you from equalising.

► We recommend that you replace your Earwear DIVE at least every four years.

**Cleaning and care:**

► Clean your Earwear DIVE after every use using soap and water to remove earwax (cerumen) and any other dirt, using a damp cloth or antibacterial wipes.

► Certain chemical substances can damage this product. Avoid contact with strong chemicals, cleaning agents or solvents.

► Your customer service associate will in-

form you about recommended care products.

► Please use the case provided to store your product.

**Warning and safety information:**

► Check the aperture of your Earwear EW 1521 before and after each use to ensure it is not blocked with cerumen, and if necessary clean it immediately.

**Caution:** If an aperture is blocked it will prevent you from being able to equalise.

► If you notice any pressure marks or experience any feeling of pressure, please check the fit of your earplug.

▶ Should you suffer any allergic reaction, such as red or itchy skin etc. please stop using your Earwear EW 1521 and seek medical advice about the cause of the problem.

▶ Keep away from toddlers. Risk of asphyxiation if swallowed.

▶ If you sustain an ear injury you should refrain from using your Earwear EW 1521 and we would recommend you seek medical advice.

▶ We recommend you complete an open water diving course before using the product underwater.

▶ We also advise you to have your ears cleaned by a medical professional before using Earwear DIVE.

▶ Should you encounter any problems with equalising, return slowly to the surface, making a safety stop if necessary. In this event, you should under no circumstances continue your dive or dive deeper.

▶ Contact with chemicals or alcohol can damage your earplugs. Additional information on this topic is available from the manufacturer on request.

▶ Earwear DIVE components may be damaged if they come into contact with sharp objects.

▶ The functionality of your earplugs may be impaired if these guidelines are not followed.

▶ Sudden or rapid removal of Earwear EW 1521 from your ear canal may damage

your eardrum.

► Please note that your Earwear DIVE may impede your perception of potential hazards and warning signals, so please pay special attention to your surroundings when using them.

**Service and warranty:**

Warranty claims shall be governed by statutory provisions. The warranty excludes all damage attributable to improper use, and non-compliance with the care instructions. In case of warranty claims please contact your customer service associate with your receipt.

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Neuroth International AG