
E A R W E A R

HUNT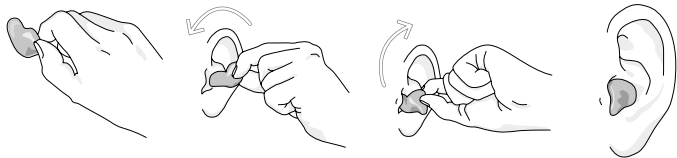

User information

EN

Dear customer,
please read these usage instructions carefully before the first use of your hearing protection and keep them in the supplied storage case. They provide important information about your product.

Protect your hearing, because noise-induced hearing impairment is incurable. Noise is unwanted sound that is not only an unpleasant nuisance but can also be a trigger for many illnesses. Noise-induced hearing impairment does not only occur at work but is increasingly often the result of leisure activities that damage hearing. Unfortunately, people often

carelessly neglect to use hearing protection. The reason for this in many cases is a lack of wearing comfort. The EARWEAR HUNT provides safe protection for our hearing and excellent wearing comfort!

General information:

- ▶ HUNT is an individually adjusted hearing protection earplug made of robust, biocompatible material that is suitable for multiple use.
- ▶ To distinguish the sides, the right earmould is marked with a dot.
- ▶ Note: Only if your hearing protection is used correctly, fits securely and is worn without interruption in the area of noise does it ensure an adequate protective effect.

- ▶ The product has been tested and approved in accordance with EN352-2:2020 and EN352-7:2020
- ▶ The type approval test was carried out by: PZT GmbH (#1974)
- ▶ The annual monitoring audit (module D) was carried out by: AUVA (#0511)
- ▶ The product fulfils the requirements of the directive 2014/53/EU of 16 April 2014, regarding the provision of radio units on the market.

NEUROTH INTERNATIONAL AG declares that EARWEAR HUNT corresponds to all the requirements of the PSA directive EU 2016/425.

- ▶ Source details for declaration of conformity: <https://www.earwear.me/service/produkt-daten>
- ▶ Source details for harmonised standards: https://eur-lex.europa.eu/eli/dec_impl/2023/941/oj
- ▶ Source for the directive PSA-VO EU 2016/425: <https://eur-lex.europa.eu/eli/reg/2016/425/oj>

Insertion

Grasp the strap holder / the ear mould with your thumb and forefinger and place the cone of the hearing protection into the auditory canal. Pull the earlobe outwards carefully and press the ear mould into the auditory

canal, if necessary with slight rotation. Follow the natural contours of your ear. Press lightly on the ear mould to ensure that it fits comfortably in the ear.

Removal

▶ Strap variant

Grasp the strap with your thumb and forefinger and pull it carefully so that the hearing protection is loosened from the auditory canal. Then remove the hearing protection from the ear.

▶ Retraction string variant

Grasp the retraction string with your thumb and forefinger and pull it carefully so that the hearing protection is loosened from the au-

ditory canal. Then remove the hearing protection from the ear.

▶ Variant without a strap

Grasp the hearing protection with your thumb and forefinger and pull it carefully so that the hearing protection is loosened from the auditory canal. Then remove the hearing protection from the ear.

Functional defects

- ▶ Check your EARWEAR HUNT regularly for damage and the need for maintenance. Do not use them if they are damaged. For a replacement, please contact the manufacturer.
- ▶ Your hearing protection product should

be checked every two years by a specialist.

- ▶ We generally recommend that you replace your hearing protection product at the latest after 3 years. (Date of recommended replacement is on the product label/case)

Cleaning and maintenance

- ▶ Clean your EARWEAR HUNT carefully after each use with the included cleaning accessories.

The HUNT must under no circumstances be washed or immersed in water or other liquids.

- ▶ This product can be negatively influenced by certain chemical substances. For further information please contact the manu-

facturer.

- ▶ You can obtain information from your customer adviser about specially recommended care products.
- ▶ Please use the supplied case for storage before and after use.

Warnings

- ▶ EARWER HUNT is suitable for shots or bangs up to 161 dB. As these values can be slightly exceeded in indoor spaces (e.g. indoor firing range), **EARWEAR HUNT may only be used outdoors (e.g. for hunting)!**
- ▶ Your EARWEAR HUNT only offers adequate protection if it is used, set and maintained in accordance with the manufactur-

er's instructions.

- ▶ In case of pressure points or a feeling of pressure, please check the fit of your hearing protection product.
- ▶ If allergic reactions occur such as skin reddening, itching etc., please discontinue use of the hearing protection and clarify the cause with a medical practitioner!
- ▶ Keep the product away from small children due to a choking hazard!
- ▶ Please note that hearing protection compromises your perceiving of possible dangers or warning signals. When using the hearing protection, therefore pay particular attention to potential dangers and warning signals in your surroundings.

- ▶ Sudden or quick removal of the hearing protection earplug from the auditory canal can damage the eardrum.

- ▶ Do not remove the earplugs by tugging on the strap!


- ▶ In case of ear injuries, the hearing protection may no longer be used and it is recommended to consult a medical practitioner without delay.

- ▶ These hearing protection earplugs should not be worn if there is a risk of the strap being snagged or tugged during wearing.

- ▶ The strap of the hearing protection must be worn over the neck and never at the front!

WARNINGS ACCORDING TO EN 352-7:2020

- ▶ EARWEAR HUNT is fitted with a device for sound insulation dependent on the noise level. The user should check that the product is functioning correctly before use.
- ▶ If distortions or interruptions are identified, the user should follow the recommendations of the manufacturer for maintenance and changing the battery.
- ▶ Performance may decline as the battery level falls. The typical usage duration for constant use is around 120 hours.

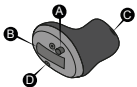- A = programme key
- B = battery compartment
- C = sound output
- D = microphone opening

Handling

By pressing briefly on the programme keys (A), you can scroll through the three programmes of EARWEAR HUNT (right side = forwards, left side = backwards)

Programme:

1 = passive insulation effect neutralises up to maximum 85dB

2=slight amplification (approx. 6dB) of quiet ambient noise up to sounds of maximum 85dB.

3=medium amplification (approx. 9dB) of quiet ambient noise up to sounds of maximum 85dB.

- ▶ By pressing on one of the programme keys (A) for longer (3-4 seconds), the ampli-

fication is deactivated for 3-4 seconds. The passive insulation effect of the earplugs remains.

- ▶ To switch on, close the battery compartment (B) with the inserted battery.
- ▶ To switch off, open the battery compartment (B)

Battery Operation

- ▶ The hearing protection is operated with replaceable batteries ZL312
- ▶ The operating time of the batteries is a maximum of 120 hours, depending on the frequency of use.
- ▶ Please replace the battery when you hear the battery warning tone.

Three warning tones = the battery is almost empty | Four warning tones = the battery is empty

- ▶ After using the hearing protection, please open the battery flap to ensure the longest possible battery lifespan.
- ▶ Please only remove the sticker on the batteries just before use.
- ▶ Please dispose of the batteries properly and on no account in the domestic waste!

Replacing the batteries:

1. Open the battery compartment fully and remove the battery.

2. Pull the protective foil off the “+” side of the new battery.

Recommendation: Wait two minutes before inserting it, so that the battery can be aired. This ensures optimal functioning.

3. Insert the new battery into the battery compartment. Ensure that the “+” sign on the battery and the “+” sign in the battery compartment are aligned.

Water and dust protection

EARWEAR HUNT is protected against the penetration of water and dust and can come into contact with sweat or get a bit wet in rain. If your hearing protection comes into contact with a lot of water and no longer works, please carry out the following steps:

5. Carefully wipe all the water off the outside of the device.

6. Open the battery compartment, remove the battery and carefully wipe any moisture in the battery compartment away.

7. Let the hearing protection dry for around 30 minutes with an open battery compartment.

8. Insert a new battery.

Do not wear your EARWEAR HUNT in the shower or for water sports. Do not submerge them in water or other liquids.

Service and warranty

In a warranty case, please contact your retailer with the receipt. The warranty entitlement is in accordance with statutory regulations. The exception is damage caused by improper use and non-compliance with the care instructions.

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Definition of terms

Values (dB)	Sound insulation in decibels, whereby for 84% of individual persons, the achieved protection is greater than or equal to the stated value.
APV (dB)	Assumed protective effect („Assumed protection value“). Average insulation value (Mf-1xSf) for the stated frequency in decibels.
Mean (dB)	Average insulation value in the stated frequency range (SNR/L/M/H) or the stated frequency ranges (frequency (Hz)) in decibels.
StD (dB)	Standard deviation for the stated sound insulation value in decibels
SNR (dB)	(„Single Number Rating“). Sound insulation value, to be deducted from the C-valued noise level.
L (dB)	Sound insulation value in a predominantly low-frequency noise spectrum (frequency <500Hz)
M (dB)	Sound insulation value in a predominantly medium-frequency noise spectrum (frequency >500Hz and frequency <2000Hz)
H (dB)	Sound insulation value in a predominantly high-frequency noise spectrum (frequency >2000Hz)

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Neuroth International AG