E A R W E A R

Monitoring

Dear customer,

Thanks you for choosing the EARWEAR MONITORING. This system was especially custom-made for you on the basis of your ear impressions and in upkeeping our quality standards.

Please read the instructions thoroughly before use and store them carefully. The instructions give you important information about your product.

The delivery of your EARWEAR MONITORING includes:

▶ Two EARWEAR MONITORING earmoulds

- incl. your identification and serial number
- ▶ 3.5 mm stereo plugs – possible option 2.5 mm
- ▶ cleaning cloth
- ▶ cleaning brush
- ▶ filter change system

Before operation

The audio output level is controlled by the audio source. You are therefore advised to switch on the system at the lowest volume and then turn it up slowly until a moderate volume is reached. Take care that the headphones are not adjusted too loudly. This can damage not only the system itself, the source, but also your ears.

Application:

- ▶ **Right/left marking:** a coloured marking distinguishes the right from the left earmould; red marking = right ear, blue marking = left ear.
- ▶ **Insertion:** your hands should be clean. In order to insert put your hand over your head and pull your ear up a bit. Take the EARWEAR MONITORING that you want to insert with the other hand and put the earmould into your ear passage with a slight turn towards the back until it sits comfortably and securely.
- ▶ **Taking it out of your ear:** take the end of the earmould and remove it from your ear with a slight turn forwards. Never pull on the cable.

Cleaning and maintenance:

- ▶ Clean the EARWEAR MONITORING when it is dirty with a soft damp cloth. To do this, use the supplied cleaning material. Do not reinsert the earmould until it is completely dry.
- ▶ Every ear produces ear wax (cerumen) which can block the acoustic openings. Therefore remove any ear wax from the surface of the cerumen protective system with the cleaning brush.
- ▶ If it is necessary to change the cerumen protective system, use the supplied exchange tool with the pull-out hook and a new filter. Turn the pull-out hook into the old filter and pull it out. Push the new filter into the acoustic opening with slight

pressure. The filter comes of the stick and stays in the acoustic opening. A cerumen protective system may only be used once. **Caution:** If a cerumen filter comes loose and gets stuck in your ear, go immediately to an ENT doctor. You will find the instructions for changing the filter on the enclosed instructions on the back of the package.

- ▶ For storing use the supplied package.

Warning:

- ▶ Please note that your EARWEAR MONITORING can be impaired by possible dangers or warning signals. When in use take care of possible dangers and warning signals in your environment.

- ▶ Keep in mind that loud levels of volume can damage your hearing.
- ▶ If your ear is injured the EARWEAR MONITORING must not be used and you are recommended to go to the nearest medical specialist.
- ▶ This system must not get into the hands of children or other persons lacking responsibility.
- ▶ Moisture, damp and heat and mechanical impacts can cause damage.
- ▶ Avoid soiling with hair spray.
- ▶ The effects of medical apparatus can cause dysfunction.

Guarantee:

The complete EARWEAR MONITORING

system is guaranteed for 12 months.
Guarantee claims comply with legal provisions. There is no guarantee for incorrect use and not complying with the maintenance instructions.
In the case of a guarantee claim contact your customer service with your invoice.

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Neuroth International AG