E A R W E A R

SLEEP

Dear customer,

Please read this user manual carefully before using your comfort product for the first time and store it in your case.

General information:

- ▶ EARWEAR SLEEP is a reuseable, individually tailored comfort product made from an unbreakable, hypo-allergenic material to reduce noise, for example when sleeping.
- ▶ Right and left side: The right side bears a red marking.
- ▶ Easy insertion and safe sealing effect can be achieved by using care ointments. Concerning this matter, please ask your customer service associate.

Insertion:

- ▶ Take your SLEEP between your thumb and index finger and with light pressure insert it into your ear canal.
- ▶ Using a slight twisting motion, insert your SLEEP deeper into your ear canal

Removal:

- ▶ Take your SLEEP between your thumb and index finger and carefully pull on it to make the earplug come out of your ear canal.

Impairment of the sleep protector's function:

- ▶ Regularly check your SLEEP for damages!
- ▶ Your comfort product should be checked annually for its correct seat in your ear by an expert.
- ▶ We generally recommend replacing your SLEEP after 4 years at most.

Cleaning and care:

- ▶ Thoroughly clean your SLEEP with lukewarm water after every use and with mild soapy water in case of strong contamination.

- ▶ Certain chemical substances can have adverse effects on this product. Avoid contact with strong chemicals, cleaning agents or solvents.
- ▶ Your customer service associate will inform you about recommended care products!
- ▶ Use the supplied case to store your products.

Warning notes:

- ▶ In case of pressure marks and feelings of pressure, please check your product's seat.
- ▶ Should you show allergic reactions such as red or itchy skin etc. please

stop using your SLEEP and clarify the reason with a specialist physician.

- ▶ Keep away from toddlers. Risk of asphyxiation when swallowed!
- ▶ Attention: The SLEEP is not a certified hearing protection and must not be used in areas with harmful noise!
- ▶ In case of ear injuries the SLEEP must not be used any more and we recommend seeking medical advice!
- ▶ Sudden or fast removal of the product from the ear canal can cause damage to the eardrums!

Service and warranty:

Warranty entitlements are based on legal provisions. Exceptions are made for damages resulting from improper use and non-compliance with the care instructions. In the event of a warranty claim, please contact your retailer.

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Neuroth International AG