
E A R W E A R

SWIM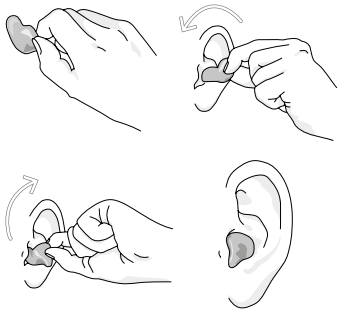

Dear customer,

Please read this user manual carefully before using the device for the first time and store it in your case. It provides valuable information on your product.

General information:

► Earwear SWIM is an individually adjusted splash water protector preventing water from entering the outer ear canal to a high degree during activities such as swimming, washing one's hair or showering.

- ▶ Earwear SWIM is not suitable for diving!
- ▶ Earwear SWIM is individually adjusted and must only be used by the person it has been adjusted to.

Application:

Take the splash water protector between your thumb and index finger and insert it into your ear canal. Use gentle pressure to move it to its final position.

Impairment of the splash water protector's function:

- ▶ Regularly check your splash water protector for damages!
- ▶ Your splash water protector should be checked annually for its correct seat in

your ear by an expert.

- ▶ Avoid jumping into the water since tightness cannot be guaranteed due to momentary distortion of the ear when hitting the water.
- ▶ We generally recommend replacing your splash water protector after 2 years at most.

Cleaning and care:

- ▶ Thoroughly clean your splash water protector with lukewarm water after every use and with mild soapy water in case of strong contamination.
- ▶ Certain chemical substances can have adverse effects on this product. Avoid contact with strong chemicals, cleaning

agents or solvents.

▶ Your customer service associate will inform you about recommended care products!

Warning notes:

▶ In case of pressure marks and feelings of pressure, please check your splash water protector's seat.

▶ Should you show allergic reactions such as red or itchy skin etc. please stop using your splash water protector and clarify the reason with a specialist physician.

▶ Keep away from toddlers. Risk of asphyxiation when swallowed!

▶ Earwear SWIM must not be used for

scuba diving! The sealing effect preventing water from entering makes the pressure equalisation required in scuba diving impossible.

▶ Toddlers should only use the splash protector under the supervision of an adult. Please make sure your toddlers don't take the otoplastics out of their ears themselves.

▶ Water sports posing the hazard of hitting the water hard with the ear are to be avoided. Momentary distortion of the ear can lead to leaking and the used splash water protector may create a loud noise caused by a diaphragm

effect which may be harmful to your hearing.

► Please note that your splash water protector may impede your perception

of possible hazards and warning signs. Therefore, pay special attention to possible hazards and warning signs in your surroundings when using your splash water protector.

► In case of ear injuries the splash water protector must not be used any more and we recommend seeking medical advice!

Service and warranty:

Warranty claims shall be governed by statutory provisions. The warranty ex-

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Neuroth International AG